


Rapid disinfection

# DESCOCEPT SPEZIAL

## Alcohol-free rapid disinfection

- Particularly suitable for alcohol-sensitive surfaces
- Strong cleaning power
- Short contact time

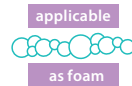

### Product description

DESCOCEPT SPEZIAL is an alcohol free ready-to-use solution for disinfection and cleaning of medical inventory, medical devices and all kinds of surfaces. Especially for alcohol-sensitive medical devices and surfaces, also suitable for acrylic glass. Based on quaternary ammonium compounds.

DESCOCEPT SPEZIAL is alcohol free, contains neither aldehydes or phenols, nor perfums.

### Range of application

**Acc. to Biocidal Products Regulation (BPR):** Disinfection and cleaning of alcohol-sensitive surfaces and medical inventory.

### Acc. to EU Medical Devices Regulation:

Disinfection and cleaning of alcohol-sensitive non-invasive medical devices.

### Application

Wipe surfaces with DESCOCEPT SPEZIAL until completely wet and allow to dry for the entire contact time.

### Application notes

DESCOCEPT SPEZIAL can be used as a wipe disinfectant or foam spray application.

### Use in wipe dispenser systems:

DESCOCEPT SPEZIAL is especially suited for

Spectrum of efficacy and contact times			30 s	1 min	2 min
<b>Bacteria and yeasts</b>					
application recommendation for surface disinfection: Bactericidal, yeasticidal*	VAH** / EN 16615	with mechanical action, dirty conditions		•	
<b>Enveloped viruses</b>					
active against enveloped viruses	RKI/DVV	incl HBV/HIV/HCV	•		
	EN 14476	clean conditions	•		
<b>Non-enveloped viruses</b>					
active against papova/SV40 viruses	EN 14476	dirty conditions			•
active against rotaviruses	RKI/DVV		•		

\* including phase 2 step 1 and phase 2 step 2 tests (quantitative suspension tests and quantitative carrier tests)

\*\* listed in the 5 min column of the VAH list

EN standards			
Antimicrobial properties			1 min
bactericidal	EN 1040	basis test	•
yeasticidal (C. albicans)	EN 1275	basis test	•
bactericidal (medical area)	EN 13727	dirty conditions	•
yeasticidal (C. albicans, medical area)	EN 13624	dirty conditions	•
bactericidal (food and industrial area)	EN 1276 / EN 13697	dirty conditions	•
yeasticidal (C. albicans, food and industrial area)	EN 1650 / EN 13697	dirty conditions	•

use with wipe dispenser systems for a convenient and user-friendly application with a proven disinfectant efficacy and standing

time of up to 28 days in combination with the DESCO/ECO Wipes dispenser system.



# DESCOCEPT SPEZIAL

Alcohol-free rapid disinfection

## Composition

100 g solution contain:  
0.5 g alkyl(C12-16)dimethylbenzylammonium chloride,  
0.25 g didecyldimethylammonium chloride.

## Expert opinions

We have extensive expert reports for all our products. Upon request we are glad to supply you with an expert report portfolio.

## Precautionary and hazard statements

Harmful to aquatic life with long lasting effects. Avoid release to the environment. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.

For professional use only.

Product	Single unit	Content	Delivery unit	REF
DESCOCEPT SPEZIAL	Spray bottle	750ml	6	00-208-0075EXP-SF
	Bottle	1L	12	00-208-010EXP
	Canister	5L	3	00-208-050EXP

CE 0482

The availability of the products and container sizes depend on a successful national registration.



Use disinfectants safely. Always read label and product information before use.

Dr. Schumacher is certified according to DIN EN 13485, DIN EN ISO9001, DIN EN ISO14001, BS OHSAS 18011 and has a validated environment management system according to EMAS and is a member of IHO, VCI, BAH, DGVS and of the DGKH. [www.schumacher-online.com](http://www.schumacher-online.com)

